Soft Disposable

Single Patient Use BP Cuff

English

Suntech Medical's Disposable Clinical Grade Blood Pressure Cuffs

Index Line
 Cuff index line must fall within range markings

Artery symbol and arrow should be placed over brachial or femoral artery

Not made with natural rubber latex

Product in compliance with Council Directive 93/42/EEC Medical Devices Directive

Symbol indicating arm circumference

Symbol indicating lot code of cuff

Symbol indicating manufacturer

Symbol indicating Caution

Indicates device sale only by or on the order of a licensed practitioner

Indicates keep dry

RANGES / COLORS

Size:	Color:	Range:
Infant	Orange	8-13 cm
Child	Green	12-19 cm
Child Long	Green	12-19 cm
Sm Adult	Turquoise	17-25 cm
Sm Adult Long	Turquoise	17-25 cm
Sm Adult Plus	Turquoise	18-29 cm
Adult	Navy Blue	23-33 cm
Adult Long	Navy Blue	23-33 cm
Adult Plus	Navy Blue	28-40 cm
Lg Adult	Burgundy	31-40 cm
Lg Adult Long	Burgundy	31-40 cm
Lg Adult Plus	Burgundy	40-55 cm
Thigh	Brown	38-50 cm

Neonatal:	Color:	Range:
Size:	Orange	3-6 cm
1	Turquoise	4-8 cm
2	Green	6-11 cm
3	Navy Blue	7-13 cm
4	Burgundy	8-15 cm

WARNINGS

Pressurization of the cuff can temporarily cause loss of function if simultaneously used with monitoring equipment on the same limb.

The cuff should not be placed on the patient's arm on the side of a mastectomy. In the case of a double mastectomy use the side of the least dominant arm.

If luer lock connectors are used in the construction of tubing, there is a possibility to connect the cuff to intravenous fluids, allowing air to be pumped into a blood vessel, potentially causing serious injury.

Following the application of the BP cuff, petechia formation (a minute reddish or purplish spot containing blood that appears in the skin) or Rumpel-Leede phenomenon (multiple petechia) on the arm, which may lead to idiopathic-thrombocytopenia (spontaneous persistent decrease in the number of platelets, associated with hemorrhagic conditions) or phlebitis (inflammation of a vein) may be observed.

Do not attach the cuff to a limb being used for IV infusions or any other intravascular access, therapy or an arterio-venous (A-V) shunt. The cuff inflation can temporarily block blood flow, potentially causing harm to the patient.

CAUTIONS

Do not use cuff if there are any signs of damage. Failure to do so could affect measurement accuracy

Minimize limb movement during the measurement.

Federal (US) law restricts this device to sale by or on the order of a licensed healthcare practitioner

The cuff should not be applied over a wound as this can cause further injury

Avoid contact with the cuff, other than that of the patient's limb, while measurement is in progress.

A compressed or kinked connection hose may cause continuous cuff pressure resulting in blood flow interference and potentially harmful injury

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