For assistance, contact:  
Pour une assistance, contactez:  
Beratung und Unterstützung erhalten Sie hier:  
Per assistenza contattare:  
Para obtener asistencia, póngase en contacto con:  
Para assistência entre em contato com:  
如需支持，请联系：

SunTech Medical, Inc.  
507 Airport Boulevard, Suite 117  
Morrisville, NC 27560 USA  
Tel: 800.421.8626  
919.654.2300  
Fax: 919.654.2301  
Email: CustomerSupport@SunTechMed.com  
Web: www.SunTechMed.com

To register this device, go to  
[www.SunTechMed.com/register](http://www.SunTechMed.com/register)

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Download operating instructions for this product by visiting [suntechmed.com/ct50-documents](http://suntechmed.com/ct50-documents). View the instructions for this product using Adobe Acrobat Reader, or any compatible PDF reader, on any computer or mobile device with an internet connection.

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### Welcome to the SunTech CT50!

The CT50 vital signs monitor easily provides reliable blood pressure (BP), SpO<sub>2</sub>, and temperature measurements for neonatal, pediatric, and adult patients.

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82-0546-00 MA Rev. B

**SunTech Medical**  
The Difference in Clinical Grade®

# Getting Started with CT50

## Introduction au CT50

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## Guia Rápido do CT50

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# CT50 Home Screen

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### CT50 Startbildschirm

### CT50 Schermo di casa

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## CT50 Tela inicial

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#### English

- Physiological alarm visual indicator LED's.
  - High level alarm: the lamp quickly flashes red.
  - Medium level alarm: the lamp slowly flashes yellow.
  - Low level alarm: the lamp lights yellow without flashing.
- LCD Touch screen
- SpO<sub>2</sub> connector
- NIBP connector
- USB connector x 2
- Power button
- Battery charging indicator LED
- Power indicator LED. Status of the LED is specified as follows:
  - Green: When the AC mains connected.
  - Orange: When the AC mains not connected and monitor is powered by battery.
  - Off: When the AC mains not connected.
- Well for 20-count Probe Cover box
- Covidien Filac 3000 temperature probe
- Print Recorder
- Grounding terminal
- Nurse call connector
- AC power connector (input)
- DC power connector (output)
- USB socket x 2
- Ethernet LAN Network connector

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#### English

- Clinician Information  
Note: In monitoring mode, you must login a clinician prior to selecting the patient.
- System Time, Date, and Network status
- Battery Status
- Device Alarm Message Bar
- Measurement Display Area
- Patient Information
- Menu Tabs: navigate the device menu.
  - MEASURE: Display vital sign parameters.
  - PATIENT
  - REVIEW: Historical patient data.
  - ALARM: Adjust alarm parameters
  - SETTINGS
- Shortcut Icons
  - Help key
  - Alarm pause key
  - Shortcut key to reset the alarm
  - Shortcut key to print
  - Shortcut key to start/stop NIBP measurement
  - Shortcut key to standby mode
  - Shortcut key to the home screen
- Save Icon: Press to save the current measurement data for the current patient.

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