

# Oscar 2 ABPM Training Checklist

Name \_\_\_\_\_

## 1. Device Programming

- Configure patient information and schedule measurements
- Show familiarity with advanced programming options
- Save and open programming templates

## 2. Patient Setup

- Explain and discuss study with patient:
  - Study purpose
  - How to respond to readings
  - How to stop reading in case of emergency
  - Sleeping with the device
  - How to return device
- Use correct cuff size
- Correctly position cuff
- Attach device to patient
- Take manual reading to assess accuracy and comfort

## 3. Retrieving Study

(Use sample patient studies if necessary)

- Download study to AccuWinPro
- Edit graph thresholds
- Edit patient information
- Add interpretive summary

## 4. Report Export

- Modify report structure in settings
- Export to PDF

## 5. Cleaning protocols

- Familiar with cleaning protocols in Device and Cuff IFU

Evaluator  
Signature \_\_\_\_\_

Date \_\_\_\_\_

## Instructor Notes

### Equipment Needed for this Exercise

- Oscar 2 M250 ABPM system
  - Monitor
  - Size 2 and 3 Orbit required, size 1-4 preferable
  - Pouch
  - Belt
  - USB-Micro cable
- 2xAA batteries
- PC with AccuWinPro installed
- Oscar 2 IFU – digital download
- Orbit cuff IFU
- Sample patient studies (stored in AccuWinPro install folder)