



BLOOD PRESSURE MODULE WALL MOUNT INSTALLATION GUIDE


1. Pieces & Parts

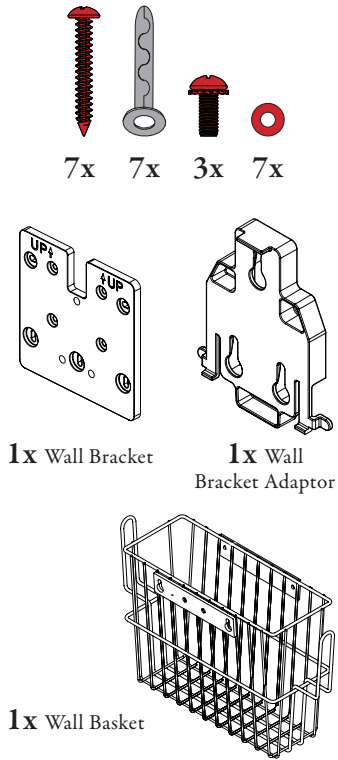


2. Attaching the Wall Bracket

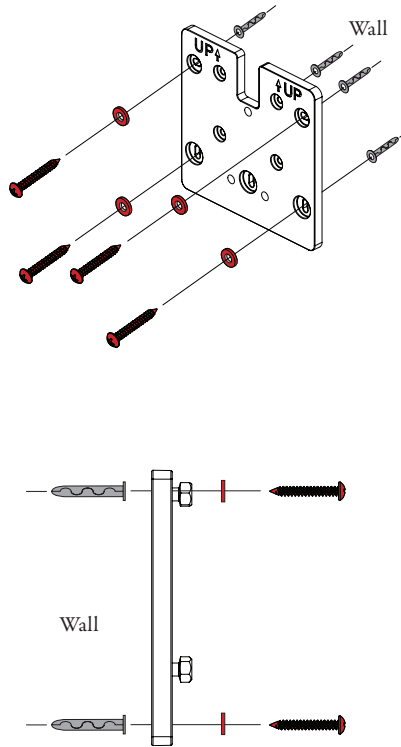


3. Attaching the Wall Bracket Adaptor

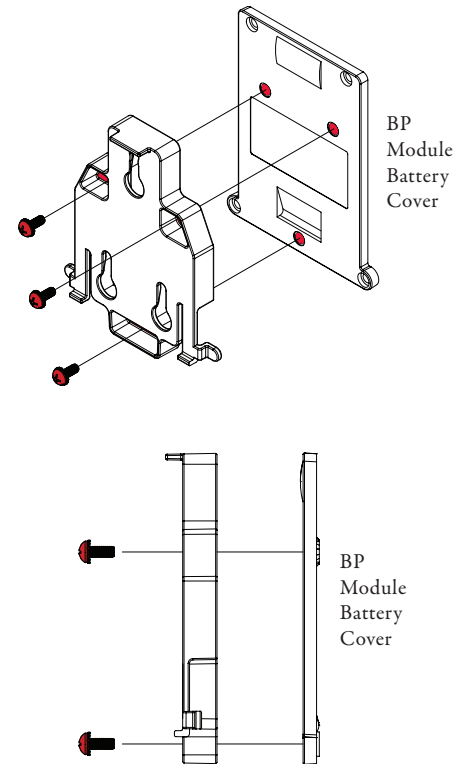


4a. Attaching the BP Module

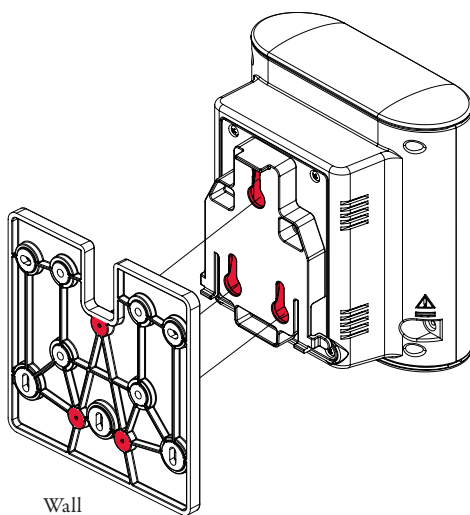

4b. Attaching the BP Module

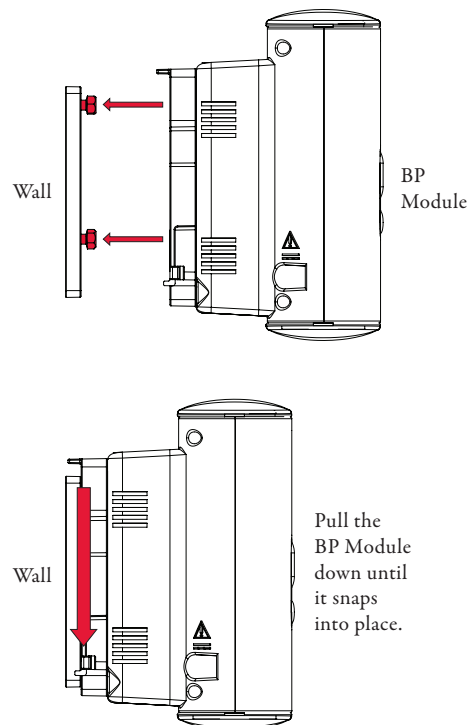


5. Attaching the Wall Basket

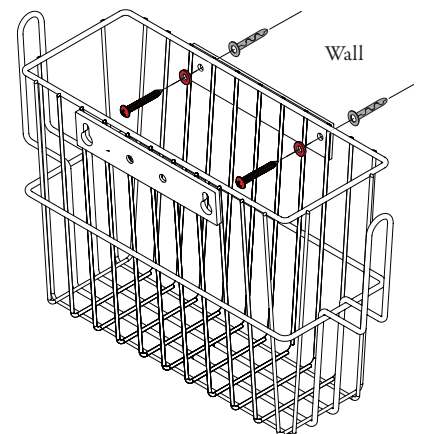

1.
Replacing a Welch Allyn Wall Aneroid


- a. Remove the screw at the bottom of the basket behind the wall aneroid.
- b. Remove the aneroid.
- c. Remove the top two screws connecting the basket to the wall.
- d. Slide the basket up and towards you to remove it from the wall.

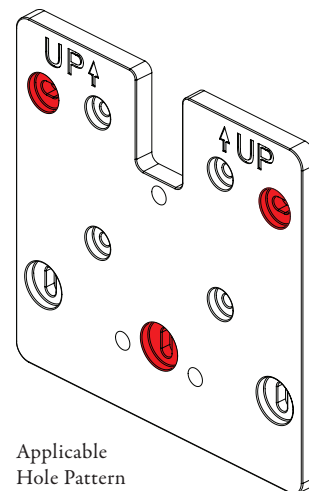

Applicable
Hole Pattern

2.
Replacing an ADC Wall Aneroid


- a. Remove the screws at the top and the bottom of the wall aneroid.
- b. Remove the aneroid.
- c. Remove the screws attaching the bracket to the wall.

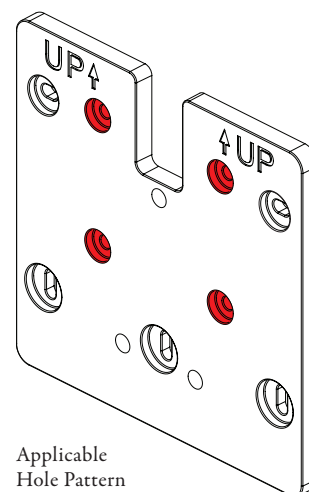

Applicable
Hole Pattern

3.
Replacing a Medline Wall Aneroid

- a. Remove the screws at the top and the bottom of the wall aneroid.
- b. Remove the aneroid.
- c. Remove the screws attaching the bracket to the wall.

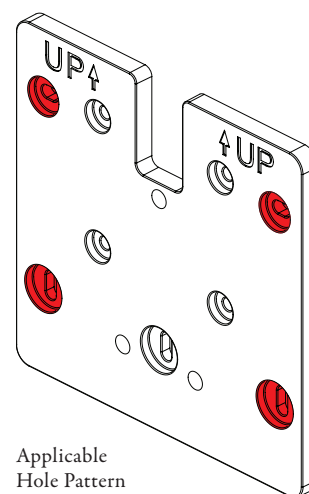

Applicable
Hole Pattern