
ENGLISH

Suntech Medical's Disposable Clinical Grade Blood Pressure Cuffs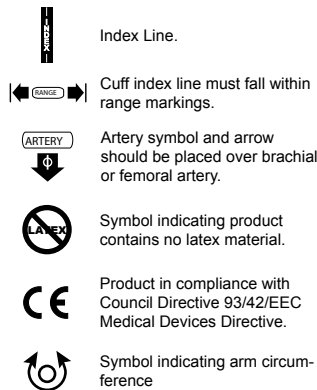

RANGES / COLORS

Size:	Color:	Range:
Thigh	Brown	38-50 cm
Lg Adult Long	Burgundy	31-40 cm
Lg Adult	Burgundy	31-40 cm
Adult Long	Navy Blue	23-33 cm
Adult	Navy Blue	23-33 cm
Sm Adult Long	Turquoise	17-25 cm
Sm Adult	Turquoise	17-25 cm
Child Long	Green	12-19 cm
Child	Green	12-19 cm
Infant	Orange	8-13 cm

Neonatal:

Size:	Color:	Range:
1	Orange	3-6 cm
2	Turquoise	4-8 cm
3	Green	6-11 cm
4	Navy Blue	7-13 cm
5	Burgundy	8-15 cm

WARNINGS & INDICATIONS

Pressurization of the cuff can temporarily cause loss of function if simultaneously used with monitoring equipment on the same limb.

Replace cuff when worn or damaged.

If patient has a mastectomy, place the cuff on the arm opposite the side of the mastectomy. If a patient has a double mastectomy, the cuff must be placed on the least dominate arm.

If a patient is undergoing intravascular access or therapy, or an arterio-venous (A-V) shunt, due to the presence of temporary interference of blood flow, do not apply the cuff on that arm. Potential injury to patient may occur.

The use of cuff while using other monitoring medical equipment could affect the accuracy or operation of said equipment.

Federal (US) law restricts this device to sale by or on the order of a physician.

Cuffs are intended for use with non-invasive blood pressure monitors only.

Follow the application instructions for use to ensure the correct size cuff for the patient. Failure to do so will adversely affect the accuracy of the reading.

If luer lock connectors are used in the construction of tubing, there is a possibility to connect the cuff to intravenous fluids, allowing air to be pumped into a blood vessel, potentially causing serious injury.

Minimize limb movement during the measurement.

Avoid contact with the cuff, other than that of the patient's limb, while measurement is in progress.

Do not compress or pinch cuff tubing.

Ensure patient's skin is intact before applying the cuff.

Promptly remove cuff from patient when monitoring is not in progress.

During frequent or extended measurement, check cuff site and limb to ensure proper blood flow. Do not apply cuff where circulation may be compromised.

APPLICATION

• Wrap the cuff around the patient's upper arm midway between the elbow and the shoulder.

• Ensure the ARTERY arrow is over the brachial artery, between the biceps and triceps muscles on the inside of the arm.

• Use the RANGE indicator with the INDEX line to check that the arm falls within the specified range for that cuff. Use appropriate larger or smaller cuff if necessary.

• Wrap the cuff snugly around the arm.

ENVIRONMENTAL CONDITIONS

Operating Ranges

Temp: 0 - 50°C

RH: 15 - 95% non-condensing

Storage Ranges

Temp: -20 - 65°C

RH: 15 - 95% non-condensing

WARRANTY

Suntech Medical warrants our blood pressure cuff products to be free from defects in material and workmanship 24 months from the original date of purchase. The warranty shall only cover out-of-box failures, as this is a Single Use product. It is not intended for repeated use.

Contact the Service Department at either of these locations to receive a Return Material Authorization number before sending any product. Cuffs should be returned to the attention of the appropriate Service Department at the addresses below.

Suntech Medical, Ltd. Europe
Oakfield Industrial Estate
Stanton Harcourt Road
Eynsham,
Oxfordshire OX29 4TS
England
TEL: 44.1 865.884 234
FAX: 44.1 865.884 235

Suntech Medical, Ltd. Europe
Oakfield Industrial Estate
Stanton Harcourt Road
Eynsham,
Oxfordshire OX29 4TS
England
TEL: 44.1 865.884 234
FAX: 44.1 865.884 235

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SunTech Medical, Inc.
SunTech Medical (Shenzhen) Co., Ltd.