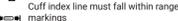
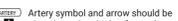
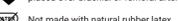
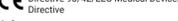
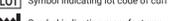
Soft Disposable Single Patient Use BP Cuff

English

	Index Line
	Cuff index line must fall within range markings
	Artery symbol and arrow should be placed over brachial or femoral artery
	Not made with natural rubber latex
	Product in compliance with Council Directive 93/42/EEC Medical Devices Directive
	Symbol indicating arm circumference
	Symbol indicating lot code of cuff
	Symbol indicating manufacturer
	Symbol indicating Caution
	Indicates device sale only by or on the order of a licensed practitioner
	Indicates keep dry
	Medical Device
	Single Patient Multi-Use

RANGES / COLORS

Size:	Color:	Range:
Infant	Orange	8-13 cm
Child	Green	12-19 cm
Child Long	Green	12-19 cm
Sm Adult	Turquoise	17-25 cm
Sn Adult Long	Turquoise	17-25 cm
Sn Adult Plus	Turquoise	18-29 cm
Adult	Navy Blue	23-33 cm
Adult Plus	Navy Blue	23-40 cm
Lg Adult Long	Navy Blue	31-40 cm
Lg Adult Plus	Burgundy	31-40 cm
Thigh	Brown	38-50 cm

Neonatal:	Color:	Range:
Size:	Color:	Range:
1	Orange	3-6 cm
2	Turquoise	4-8 cm
3	Green	6-11 cm
4	Navy Blue	7-13 cm
5	Burgundy	8-15 cm

WARNINGS
Pressurization of the cuff can temporarily cause loss of function if simultaneously used with monitoring equipment on the same limb. The cuff should not be placed on the patient's arm on the side of a mastectomy. In the case of a double mastectomy use the side of the most dominant arm.

If Luer lock connectors are used in the construction of tubing, there is a possibility to connect the cuff to measurement devices, allowing air to be pumped into a blood vessel, potentially causing serious injury.

Following the application of the BP cuff, petechia formation (a minute reddish or purplish spot containing blood that appears in the skin) or Rumpel-Leede phenomenon (multiple petechia) on the arm, which may lead to idiopathic thrombocytopenia (spontaneous persistent decrease in the number of platelets, associated with hemorrhagic conditions) or phlebitis (inflammation of a vein) may be observed.

Do not attach the cuff to a limb being used for IV infusions or any other intravascular access, therapy or an arterio-venous (A-V) shunt. The cuff inflation can temporarily block blood flow, potentially causing harm to the patient.

CAUTIONS

Do not use cuff if there are any signs of damage. Failure to do so could affect measurement accuracy. Minimize limb movement during the measurement. Federal (US) law restricts this device to sale by or on the order of a licensed healthcare practitioner.

The cuff should not be applied over a wound as this can cause further injury.

Avoid contact with the cuff, other than that of the patient's limb, while measurement is in progress.

A compressed or kinked connection hose may cause continuous cuff pressure resulting in blood flow interference and potentially harmful injury to the patient. Using an incorrect cuff size could result in erroneous and misleading BP measurement results.

APPLICATION

Follow the application instructions for use to ensure the correct size cuff for the patient. Failure to do so will adversely affect the accuracy of the reading.

- Place open cuff around the inner portion of the upper arm (or thigh).
- Align artery symbol **ARTERY** to the brachial (or femoral) artery.
- Use the **RANGE** indicator with the **INDEX** line to check that the limb falls within the specified range of that cuff. If it does not, select a cuff that better accommodates the limb circumference.
- Wrap the cuff snugly around the arm (or thigh).

Promptly remove cuff from patient when monitoring is not in progress.

Disposal: Please dispose of cuff according to local regulations

	Operating Ranges
	Temp.: 0 - 50° C
	RH: 15 - 95% non-condensing
	Storage Ranges
	Temp.: 20 - 65° C
	RH: 15 - 95% non-condensing

WARRANTY
SunTech Medical warrants our blood pressure cuff products to be free from defects in material and workmanship 24 months from the original date of purchase. The warranty shall only cover out-of-box failures, as this is a Single Use product. It is not intended for repeated use.

Contact the Service Department at each of these locations to receive a Return Material Authorization number before sending any product. Cuffs should be returned to the attention of the appropriate Service Department at the addresses below.

DISPOSABLE CUFF INTENDED USE STATEMENT

The Disposable Blood Pressure cuff is intended to be used with a non-invasive blood pressure measurement system to determine blood pressure parameters on neonate, pediatric and adult patients.

SunTech Medical, Inc. (SunTechMed.com)
507 Airport Boulevard, Suite 117
Morrville, NC 27560-8200 USA
Tel. + 1 919 654 2300
Fax: + 1 919 654 2301

SunTech Medical, Inc. (SunTechMed.com)
507 Airport Boulevard, Suite 117
Morrville, NC 27560-8200 USA
Tel. + 1 919 654 2300
Fax: + 1 919 654 2301

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